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The screenshot shows the 'CS1000 Config' application window. It is divided into two main sections: 'Device Initialization' and 'NET & Server Setting'.

**Device Initialization:**

- 1. Conform the NET-Router support DHCP function. Connect RJ-45 network cable to CS1000, then power on.
- 2. This Computer must connect to same NET-Router, and allow UDP port 60000 pass. It's recommended to close the firewall.
- 3. Refresh the list and select the device to configure.

A 'Refresh' button is located to the right of the instructions. Below the instructions is a table with the following data:

Name	IP	Work Type	Server IP	Port
CS1000 LAN	172.30.33.2	TCP CLIENT	172.30.33.1	9002

**NET & Server Setting:**

- MAC Address: 84-C2-E4-F7-A8-8B (with a 'Copy' button)
- DHCP Function:  ON  OFF
- Warning! Wrong info will lead device unusable.**
- IP Address: 172.30.33.2
- Gateway Address: 172.30.10.115
- Subnet mask: 255.255.0.0
- Connect to Server IP: 172.30.33.1
- Connect to Server Port: 9002
- If DHCP off, only can connect the router with same gateway. Or re-config under this router.**
- 'Start Config' button

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The screenshot shows a web browser window with the title '利用者管理' (User Management). The browser's address bar shows the URL 'http://localhost:8080/'. The page has a light green background. On the left side, there is a vertical menu with the following items: 'メニュー' (Menu), '利用者管理' (User Management), '部屋管理' (Room Management), 'メール通知' (Email Notification), 'ELMO QBIC CLOUD 連携管理' (ELMO QBIC CLOUD Integration Management), and 'ログアウト' (Logout). The 'ログアウト' item is highlighted. In the top right corner, there is a search bar with the text '利用者' and a '検索' (Search) button. Below the search bar is a table with the following columns: '氏名▲' (Name▲), '部屋▲' (Room▲), '機器No▲' (Device No▲), '保護者▲' (Guardian▲), '保護者連絡先▲' (Guardian Contact▲), '離床無効▲' (Bed Leaving Ineffective▲), and 'カメラ番号▲' (Camera No▲). The table contains one row with the following data: '○○○○', '101号室', '9520035', an empty cell, an empty cell, 'OFF', and an empty cell. Below the table, the number '1' is displayed. At the bottom of the page, there are four buttons: '登録' (Register), '削除' (Delete), '編集' (Edit), and 'CSV出力' (CSV Export). The browser's status bar at the bottom right shows '100%' zoom level.

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The screenshot shows a web browser window titled "CSO オプション バージョン". The main content area is divided into two sections. On the left, there is a grid of 12 room status cards, each representing a room (e.g., 101号室, 102号室, etc.) with a patient's name (represented by four circles) and a status indicator (e.g., 0, 70, 13). On the right, there is a "ヘルプ モニター画面" (Help Monitor Screen) overlay. This overlay contains several sections: "睡眠ステータス画面" (Sleep Status Screen) with a "睡眠レポートの印刷ボタン" (Print Sleep Report Button); "トレンドグラフを表示" (Show Trend Graph); "ステータス一覧" (Status List) with icons for "離床" (Out of Bed), "手動離床" (Manual Out of Bed), "起き上がり" (Getting Up), "覚醒" (Awake), "睡眠" (Sleep), "浅い睡眠" (Light Sleep), and "深い睡眠" (Deep Sleep); "バイタルデータ・2週間の睡眠遷移の画面" (Vital Data - 2-week Sleep Transition Screen); and "睡眠クオリティ" (Sleep Quality) and "ゆらぎ健康度" (Stability Health Degree) with smiley face icons and arrows indicating "良い" (Good).

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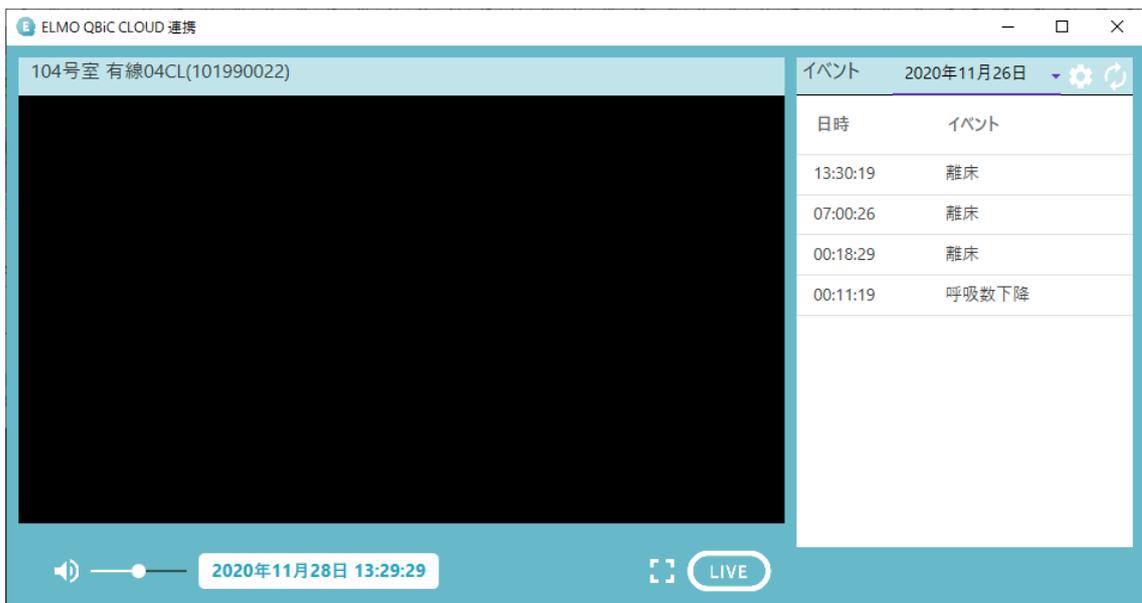
The screenshot shows the ELMO QBiC CLOUD interface. On the left is a video player for room 104 (104号室 有線04CL(101990022)). The video shows medical equipment. On the right is an event log for 2020年11月28日. The log has columns for '日時' (Date/Time) and 'イベント' (Event). The event at 13:30:17 is highlighted.

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The 'イベント設定' (Event Settings) dialog box is shown. It has a title bar with 'イベント設定' and a close button 'x'. Below the title bar is the label 'イベント設定'. There is a list of event types with radio buttons next to them:

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At the bottom of the dialog are two buttons: 'OK' and 'キャンセル'.

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